



INVACARE®

Bed Rail Selection Chart



Yes, you can.®


As the leading producer of bed systems and components (bed, mattress and rails) to the home and long-term care industries, Invacare strives to ensure patient safety. The importance of proper patient assessment, correct installation and the ongoing review and maintenance of the complete bed system should never be overlooked. With millions of bed systems in both home and long-term care environments, awareness of the potential for entrapment is vital.

Bed Rail Entrapment can occur from issues with the bed, the rail or the mattress. Please note that replacing these rails will only address certain zones for potential entrapment. **It is up to you, the provider, to ensure that your bed systems meet the FDA's guidelines. As such, you will need to evaluate your inventory of beds and mattresses as well for compliance with the guidelines, especially as the three components (bed, rail and mattress) work together as a system.**


Bed rail designs have changed over the years. Our goal is to inform you of these changes and request that you review your inventory of Invacare bed rails. If you currently use an older design, we ask that you stop using these and replace them with a newer design. The following pages depict some of the options available to you.



OUT DATED

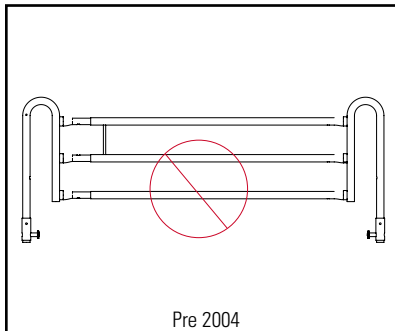

IMPROVED DESIGN

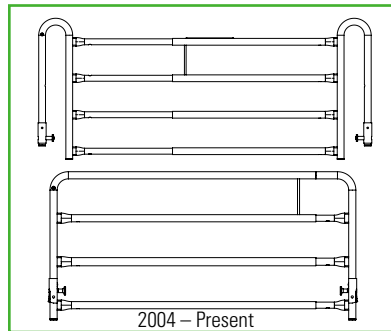



Full Length Rails

Potential Part Numbers: 6628, 6628-TSS

Replace with 6628, 6628-TSS, or 6629

Note: pre-2004 rails had 3 cross bars;
2004 to present rails have 4 cross bars

Note: 6628 Rails utilize 1" tubing

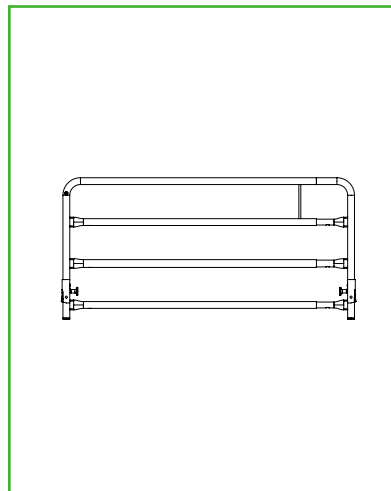
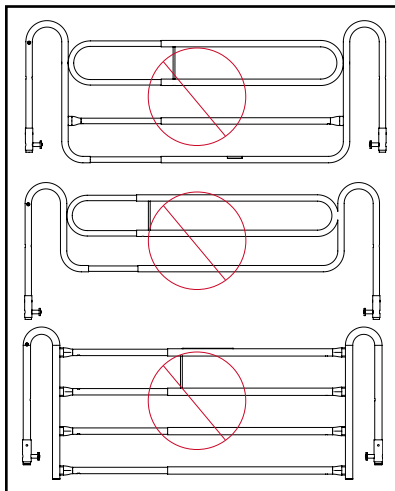


Potential Part Numbers: 6650

Replace with 6629

Note: 3 different designs exist

Note: 6629 Rails utilize 7/8" tubing

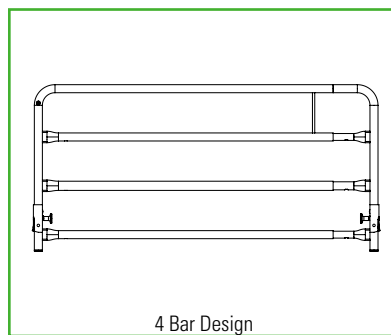
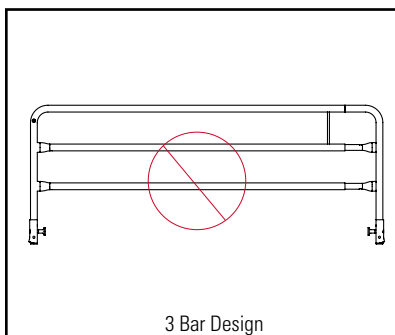


Potential Part Numbers: 6629

Replace with part number 6629

Note: pre 2004, rails had 3 cross bars

Note: 2004 to present, rails have 4 cross bars

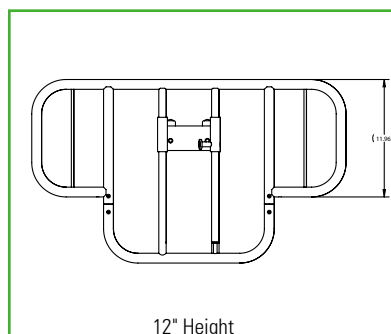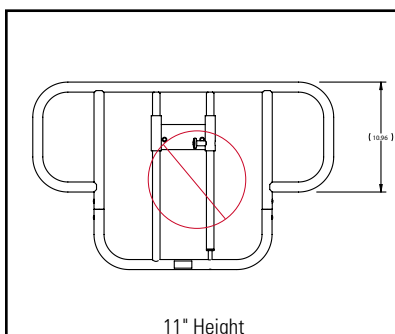


Half Length Rails

Potential Part Numbers: 6630 pre 2003


Replace with 6630 current production

Note: pre 2003 rails had different dimensions
and no spindle; from 2003 to present,
dimensions have changed and an additional
spindle was added to close the gap.

OUT DATED

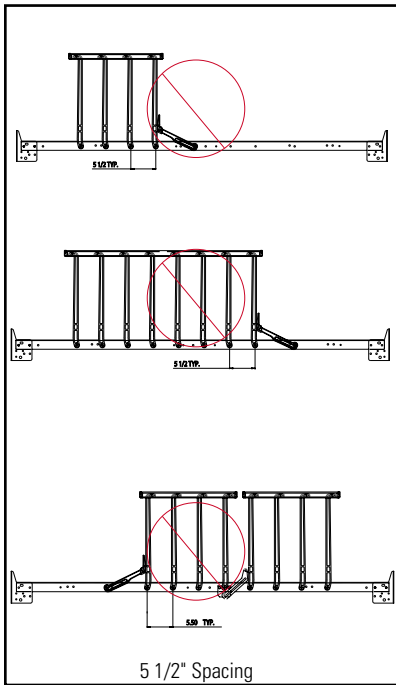

IMPROVED DESIGN


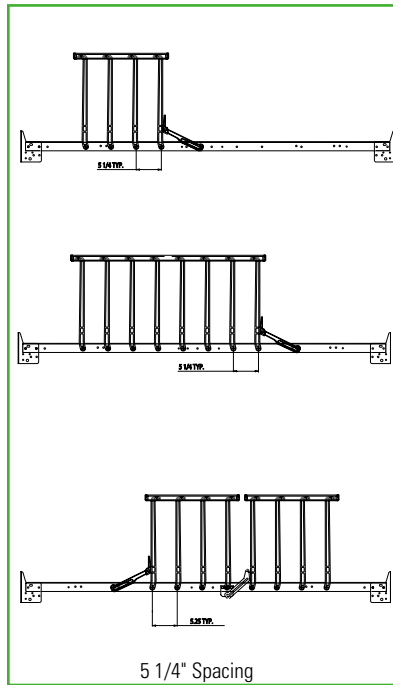


Potential Part Numbers: IH5744, IH5748, and IH5749

Replace with IH6803A, IH6808A, IH6815A, IH6816A

Note: If you have rails with a 5 1/2" spacing you should replace these with a different rail.


Note: Nylasar rails manufactured before June, 2005 had a 5 1/2" spacing between the rail attachment bolts. After June, 2005 the spacing was decreased to 5 1/4". While 5 1/4" spacing rails are suggested for use, they are no longer manufactured. If you need to replace 5 1/2" rails and do not currently have 5 1/4" rails, please replace with one of the following: IH6801A, IH6803A, IH6815A, IH6808A

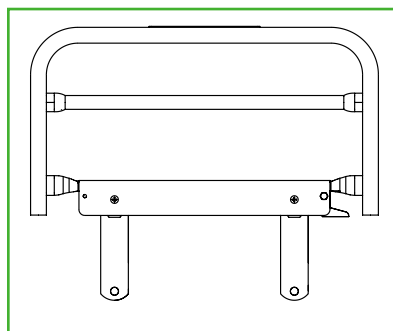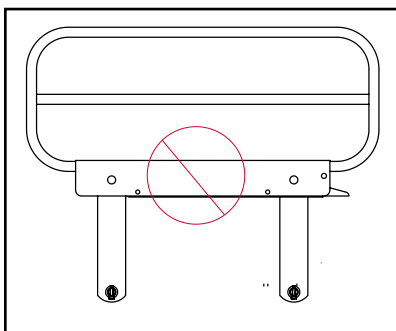


Potential Part Numbers: IH5822, IH6802, and IH6803

Replace with IH6803A



OUT DATED

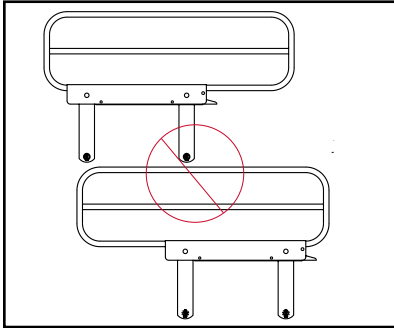



IMPROVED DESIGN

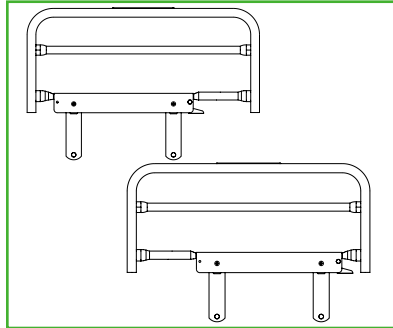

Potential Part Numbers: IH5820, IH5830, IH5831, IH6805, IH6815, IH5821 and IH6806
Replace with IH6815A and IH6816A

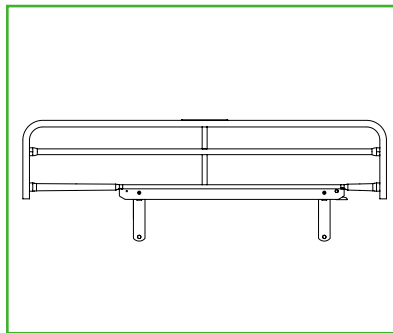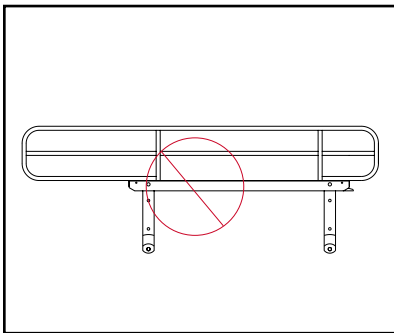

Potential Part Numbers: IH5813, IH6807
Replace with IH6808A

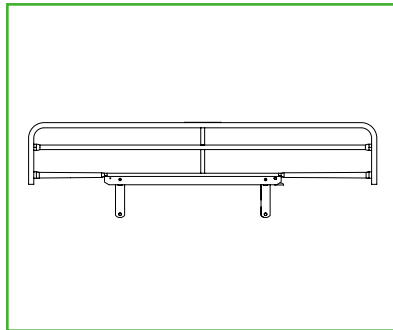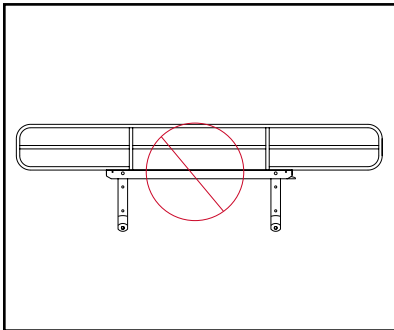


Potential Part Numbers: IH5814, IH6800
Replace with IH6801A




INVACARE CONTINUING CARE DIVISION
Carroll Healthcare Specific

OUT DATED

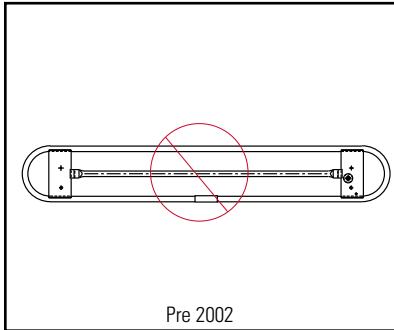

IMPROVED DESIGN


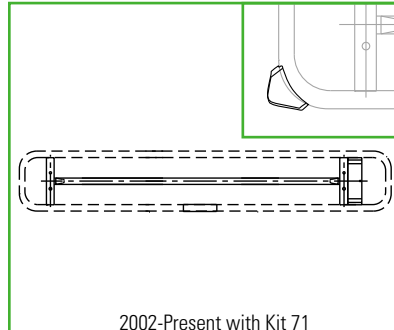


Potential Part Numbers: IHARRLFUL, IHECRLFULL, IHARRLPAR, IHECRLPAR

Replace with current production in addition to Kit 71

Note: Use corner adapter kit to modify your rails

Note: Use only with Invacare® Solace® Prevention 2000 Series and higher mattresses

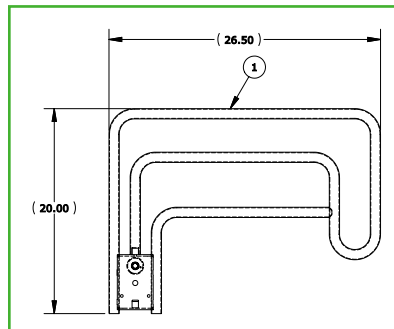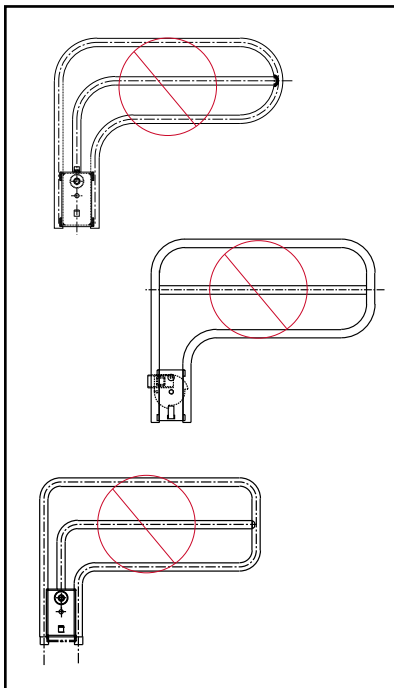

Potential Part Numbers: IH5814, IH6800

Replace with current production if your rail looks like one pictured on the left side

**Invacare Corporation
USA**

One Invacare Way
Elyria, Ohio 44035-4190
(800) 333.6900

Canada

570 Matheson Blvd. E.,
Unit 8
Mississauga, Ontario
L4Z 4G4 Canada
(800) 668.5324

©2011

Invacare Corporation.

All rights reserved.

Trademarks are identified
by the symbols [™] and [®].

All trademarks are owned
by or licensed to

Invacare Corporation
unless otherwise noted.

Specifications are subject
to change without
notification.

Form No. 11-122

110512



Yes, you can.®